

*H*Anatomic *Hygiene seat*

Care Instructions:

The cushion, (the blue material closest to the body) is hand washed with a soft sponge at 40° C max. The cushion must air dry at indoor temperature, do not place it close to any heat source. **Do not dry it in a drying cabinet or a tumble-drier or it may lose its shape.** The plastic part of the seat may be washed with a sponge or soft brush. Use a light detergent and, if necessary, a disinfectant.



***If you aren't satisfied with the seat please contact the responsible occupational therapist or Anatomic SITT.
(you find this information at the label "specialmade product").***



Box 6137, SE - 600 06 NORRKÖPING, Phone +46 (0) 11-16 18 00, fax +46 (0)11-16 20 05

www.anatomicsitt.com



*H*Anatomic *Hygiene seat*

Care Instructions:

The cushion, (blue material closest to the body) is hand washed with a soft sponge at 40° C max. The cushion must air dry at indoor temperature, do not place it close to any heat source. **Do not dry it in a drying cabinet or a tumble-drier or it may lose its shape.** The plastic part of the seat may be washed with a sponge or soft brush. Use a light detergent and, if necessary, a disinfectant.



***If you aren't satisfied with the seat please contact the responsible occupational therapist or Anatomic SITT.
(you find this information at the label "specialmade product").***



Box 6137, SE - 600 06 NORRKÖPING, Phone +46 (0) 11-16 18 00, fax +46 (0)11-16 20 05

www.anatomicsitt.com

