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## ENG – SittBelt must be mounted and adapted by medical/qualified people.

- 1) The SittBelt should be mounted on the wheelchair in the fix points. Look for a tube under or along the seat plate.
- 2) Run the strap through the enclosed Tri-glide buckle, pic. 4. If the straps are too long, cut the superfluous strap and singe the end.
- 3) Adjust the pull direction by tightening respective strap according to pic. 2.
- 4) Tighten the belt on both sides of the buckle in order to get it centered. Tighten to a suitable pressure. Release the pressure if the user feels discomfort.
- 5) The belt cushions are laterally adjustable. Adjust them to receive an optimum pressure relief.

## Supervisory control (check at every new adjustment)

- The pull direction, pic. 2 • The pressure of the strap • The position of the belt cushions.

**Washing instructions** Support Eksklusiv 400, softwash // Support Soft 600, softwash Do not dry in a tumbler or drying cabinet.

**Guarantee** 1 year. Only use spare parts delivered by Anatomic SITT AB.

**Check the user** continuously to discover any redness of the skin that remains. If such arises, you must immediately stop using the belt.

**Wear control** Check the straps, (especially at the fix points), and the belt cushions to discover any signs of wear.

**The product fits** the seating systems Zitzi Delfi, Zitzi Delfi Pro, Zitzi Sharky & Anatomic SITS. For other combinations, contact Anatomic SITT AB.

**Recycling** A worn out part or product should be disassembled and discarded. Separate and sort after material type.