

Manufactured by Anatomic SITT AB
 BOX 6137 SE-600 06 NORRKÖPING
 SWEDEN - Phone +46 (0) 16 18 00



Sternum support & Sternum vest


Sternum serie should be used together with the Anatomic SITT SittBelt for users who has a tendency to fall forwards with the sternum and shoulders.



Wrong direction



Right direction

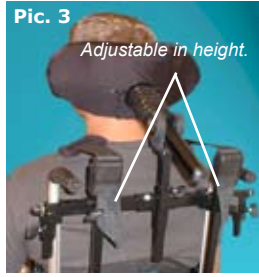

Adjustable in height.

"holder between handles"

lower attachments- **Sharky**

lower attachments - **Delfi**



Pic. 7

Tri-Glide:



Make sure that there is enough space for the neck.

Sternum support should be mounted by a medical / qualified therapist only

Mounting on a seating system or a wheelchair:

- 1) First mount the Anatomic SITT positioning SittBelt, (See separate manual), for an optimal correct pelvis position.
- 2) Mount the lower attachments of the Sternum support diagonally downwards with the enclosed Tri-glide buckle.
- 3) Mount the upper attachments in a suitable place with the enclosed buckles:

Alt. 1: When mounting on the stable back - Screw on the enclosed straplock (Pic. 2)

Alt. 2: When mounting around the frame tubes - Run the strap through the enclosed Tri-glide buckle (Pic. 7)

Alt. 3: When mounting the "hammock back" - Use the "holder between handles", item 7022 + 3719 (Pic. 3)

The pull direction of the upper attachments:

The direction of the upper attachments should be at the same level as the shoulders to prevent the shoulders from being pressed down (pic. 4 - 5). If necessary, use raising stays (Item 3715).

Adjustment of the Sternum Support:

- 1) Tighten the upper straps of the Sternum Support and adjust the Sternum plate with the four attachments (pic. 4).
- 2) Make sure that there is enough space for the neck (Pic. 4).

Supervisory control: Check at every new adjustment:

- the straps directions pic 2-3 // - the pressure of the strap // - The space for the neck // - The position of the belt cushions.

Washing instructions: 60°, softwash. Do not dry in a tumbler or drying cabinet.

Guarantee: 1 year. Only use spare parts delivered by Anatomic SITT AB.

Check the user continuously to discover any redness of the skin that remains. If such arises, immediately stop using the harness.

Wear control: Check the straps, (especially at the fix points), and the belt cushions to discover any signs of wear.



Box 6137, SE- 600 06 NORRKÖPING
 Tel. 011-16 18 00, Fax 011-16 20 05
 www.anatomicsitt.com



Viengevej 4, 8240 RISSKOV
 Tel. 86- 17 01 74, Fax: 86- 17 01 75
 www.anatomicsitt.com



Postboks 92 Nyborg, N-5871 BERGEN
 Tel. 55- 39 20 80, Fax 55- 39 20 90
 www.anatomicsitt.com



Postboks 628, N-1411 KOLBØTN
 Tel. 66- 81 60 70, Fax 66- 81 60 71
 www.hm-spes.no