VÄRMEFODER

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Heating cushion

Do not expose the battery or the charger to moisture

Heating cushion

The cushion has an built-in tempetureregulator from +27°C to +32°C. The insulation tolerates low/high tempetures and doesn't get affected by sunlight, humidity or chemicals. The material is flameproof (FMVSS 302). The heating cushions are entirely ventilated to avoid condence.

Battery

The battery contains lead, and may not be thrown with ordinary garbage. When the battery is worn-out, leave it to recycling/ destruction. The battery is rechargeable and can be recharged with the enclosed charger.

Charging the battery

1. Connect the charger to the outlet 230V, 50 Hz.
2. Connect the charger to the control-cover.
3. Make shore the red light goes on, which indicates that the charging has started.
4. Let the battery be connected until the light turns off, which indicates that charging is finished.
5. Remove the cord from the outlet.
6. Disconnect the charger from control-cover.

Usage

Only to be charged indoors. Do not expose the charger for rain. The charger can be left connected, it swithes to maintainence when fully loaded.

Only to be cleand with a damn rag alt. foam for textile wash the heating cushion has to be entirely dry before connection. (The heatingplates are removeable from the lining for washing on Zitzi Delfi and Zitzi Sharky)

Fuse

There is a glasfuse at the bottom of the control-cover.