Tillverkare / Manufacturer
Anatomic SITT AB
Box 6137, SE-60006 Norrköping
SWEDEN - Phone +46 (0)11-16 18 00

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ENG – SittBelt must be mounted and adapted by medical/qualified people.

- 1) The SittBelt should be mounted on the wheelchair in the fix points. Look for a tube under or along the seat plate.
- 2) Run the strap through the enclosed Tri-glide buckle, pic. 4. If the straps are too long, cut the superfluous strap and singe the end.
- 3) Adjust the pull direction by tightening respective strap according to pic. 2.
- 4) Tighten the belt on both sides of the buckle in order to get it centered. Tighten to a suitable pressure. Release the pressure if the user feels discomfort.
- 5) The belt cushions are laterally adjustable. Adjust them to receive an optimum pressure relief.

Supervisory control (check at every new adjustment)

- The pull direction, pic. 2 • The pressure of the strap • The position of the belt cushions.

Washing instructions Support Eksklusiv 400, softwash // Support Soft 600, softwash Do not dry in a tumbler or drying cabinet.

Guarantee 1 year. Only use spare parts delivered by Anatomic SITT AB.

Check the user continuously to discover any redness of the skin that remains. If such arises, you must immediately stop using the belt.

Wear control Check the straps, (especially at the fix points), and the belt cushions to discover any signs of wear.

The product fits the seating systems Zitzi Delfi, Zitzi Delfi Pro, Zitzi Sharky & Anatomic SITS. For other combinations, contact Anatomic SITT AB.

Recycling A worn out part or product should be disassembled and discarded. Separate and sort after material type.