
Vrighthållare / Vristholdere

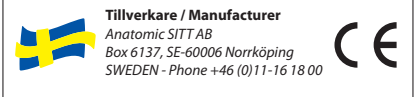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ENGLISH

Support Ankleholder provides a firm support to prevent an unintentional supination and pronation of the foot.

Mounting on the seat system / wheelchair: Support Ankleholder must be mounted and adjusted by medical / qualified people.

- 1) Mark on the foot plate where the attachments of the straps should be. The attachments should be as close to the foot as possible.
- 2) Bore a hole in the foot plate at the markings and screw up the straps with the enclosed screw.
(To obtain the right draw direction the straps should be mounted as close to the foot as possible).

Adjustment of the Ankleholder:

- 1) Mount the Ankleholder round the ankle joint with the opening forward and tighten the velcro closing.
- 2) Adjust the height of the Ankleholder at the buckle on the sides of the foot.

Supervisory control : Check at every new adjustment: * the pressure of the strap so that no stasis will occur.

* that the neoprene rubber has not "creased". Control continuously the user to discover any redness of the skin that remains.

If this would arise you must immediately stop using the Ankleholder.

Wear control: Control regularly the straps, specially at the attachments. For further information, see the product paper.

Washing instructions: Support Eksklusiv 40°, softwash // Support Soft 60°, softwash Do not dry in a tumbler or drying cabinet.

Guarantee: 1 year. Only use spare parts delivered by Anatomic SITT AB.

The product fits the seating systems Zitzi Delfi, Zitzi Delfi Pro, Zitzi Sharky & Anatomic SITS. For other combinations, please contact Anatomic SITT AB.

Recycling: A worn out part or product should be disassembled and discarded. Separate and sort after material type.

Box 6137, SE- 600 06 NORRKÖPING
Tel. 011-16 18 00, Fax 011-16 20 05
www.anatomicsitt.com

Viengevej 4, 8240 RISSKOV
Tel. 86- 17 01 74, Fax: 86- 17 01 75
www.anatomicsitt.com

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