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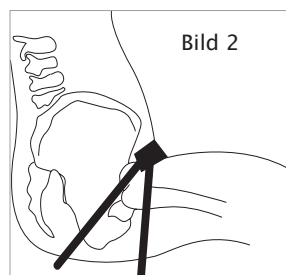
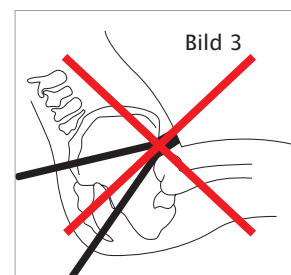
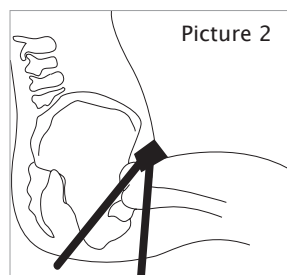
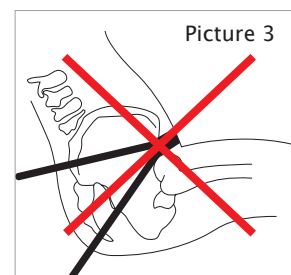
Picture 4

Tri-glide

The product must be mounted and adapted by medical/qualified people.

- 1) The belt is mounted in the chair mounting points. Look for a tube going under or at the side of the seat plate.
- 2) Runt the strap through the Tri-glide buckle shown in Figure 4. If the straps are too long, cut the strap and burn the end..
- 3) Adjust the pulling direction by tightening the straps, picture 2.
- 4) Tighten the belt on both sides of the buckle to get it centered. If the patient becomes uncomfortable release the pressure.
- 5) The cushions are adjustable sideways. Adjust them to provide optimal pressure relief.

Supervisory control (check at every new adjustment)

- The pulling direction, picture 2 • The pressure of the strap • The position of the belt cushions.

Daily care

Wipe with a damp cloth and / or disinfectants. Visual oversight of the seams, straps and buckles, making sure it works properly.

Washing instructions

60°, soft wash. Do not tumble dry. Drying cabinet, low temperature, we recommend, however, hang drying (dries faster). For longer durability wash in laundry bag.

Guarantee

1 year. Only use spare parts delivered by Anatomic SITT AB.

Controls

Make continuous checks of the user to discover any skin redness that remains. If this occurs, immediately stop using the product. Check the straps, (especially at the points of attachment); and belt cushions to discover any wear.

Service

In order for the product to work properly and safely, you need to check it regularly. The need must be assessed case by case, depending entirely on the use and wear. Work should always be professional and qualified staff. If you are unsure, you are always welcome to contact Anatomic SITT AB.

Allowed adaptations

Cutting the straps. Each time the product is changed in any way you have to assess whether it is an adaptation or customization. An adaptation is a change made in the context of the product's CE Mark. Everything else is a customization. Adjustments may be made only by the person who is considered qualified for the task and should always be performed in a professional manner. The CE mark means that the product meets all relevant European safety requirements.

The product fits Anatomic SITT's seating systems as well as other brands of wheel chairs and seating systems. For other combinations, contact Anatomic SITT AB.

Recycling

A worn out part or product should be disassembled and discarded. Separate and sort after material type.

This product is intended to securely position a user. The product should not be used as safety belt during transport, as restraint of a person or in any other manner that could cause injury.

NOTE! The prescriber of the medical device, or someone with equivalent skills and knowledge of the patient and the medical device, must always perform a risk analysis before use. If this analysis shows that there may be risk of serious injury, the patient must never be left unattended when the medical device is used.

The Support Serie is not intended to redplace health care staff supervision.

Box 6137, SE- 600 06 NORRKÖPING
Tel. +46 11-16 18 00, Fax +46 11-16 20 05
www.anatomicstt.com

Viengevej 4, 8240 RISSKOV
Tel. +45 86- 17 01 74, Fax: +45 86- 17 01 75
www.anatomicstt.com

Håndverksvein 8, N-1405 Langhus
Tel. +47 64- 91 80 60, Fax +47 64- 91 80 66
www.bardum.no