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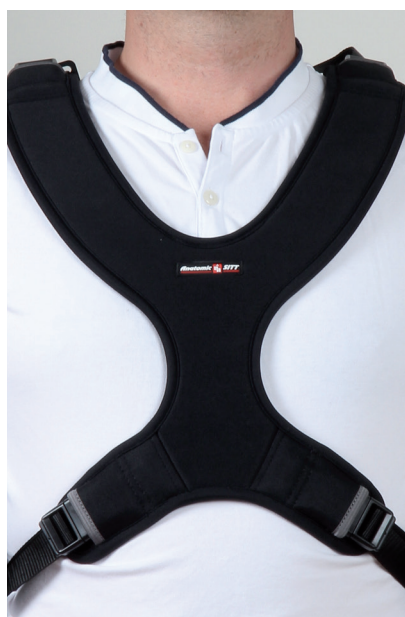
The Support Serie is developed and CE-marked to fit wheelchairs/seating systems on the European Medical Aids Market.

Make sure that there is enough space for the neck.

The harness MUST be used in combination with a hip belt!

The product should be mounted by a medical / qualified therapist only.

The Support Serie is not intended to replace health care staff supervision.

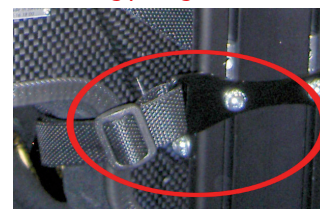
Right pulling direction

Wrong pulling direction

Lower attachments  
Zitzi Delfi Pro

Lower attachments  
Zitzi Sharky Pro

Tri-glide

#### Mounting on a seating system or a wheelchair:

- 1) First mount the Anatomic SITT positioning SittBelt, (See separate manual), for an optimal pelvis position.
- 2) Then mount the positioning strap, H-strap, with the right pull direction to position and possibly correct the pelvis optimally.
- 3) Mount the lower attachments of the strap diagonally downwards with the enclosed Tri-glide buckle.
- 4) Mount the upper attachments in a suitable place with the enclosed buckles:

Alt. 1: When mounting on the stable back - Screw on the enclosed straplock

Alt. 2: When mounting around the frame tubes - Run the strap through the enclosed Tri-glide buckle

Alt. 3: When mounting the "hammock back" - Use the "holder between handles", item 7022 + 3719

#### The pull direction of the upper attachments:

The direction of the upper attachments should be at the same level as the shoulders to prevent the shoulders from being pressed downwards. If necessary, use raising stays (Item 3715).

#### Adjustment of the Sternum support:

- 1) Tighten the upper straps of the Sternum support.
- 2) Make sure that there is enough space for the neck

#### Supervisory control (check at every new adjustment)

- The pulling direction • The pressure of the strap • Space around the neck • The position of the belt cushions.

#### Daily care

Wipe with a damp cloth and / or disinfectants. Visual oversight of the seams, straps and buckles, making sure it works properly.

#### Washing instructions

60°, soft wash. Do not tumble dry. Drying cabinet, low temperature, we recommend, however, hang drying (dries faster). For longer durability wash in laundry bag.

#### Guarantee

1 year. Only use spare parts delivered by Anatomic SITT AB.

#### Controls

Make continuous checks of the user to discover any skin redness that remains. If this occurs, immediately stop using the product.

Check the straps, (especially at the points of attachment); and belt cushions to discover any wear.

#### Service

In order for the product to work properly and safely, you need to check it regularly. The need must be assessed case by case, depending entirely on the use and wear. Work should always be professional and qualified staff. If you are unsure, you are always welcome to contact Anatomic SITT AB.

#### Allowed adaptations

Cutting the straps. Each time the product is changed in any way you have to assess whether it is an adaptation or customization. An adaptation is a change made in the context of the product's CE Mark. Everything else is a customization. Adjustments may be made only by the person who is considered qualified for the task and should always be performed in a professional manner. The CE mark means that the product meets all relevant European safety requirements.

**The product fits** Anatomic SITT's seating systems as well as other brands of wheel chairs and seating systems. For other combinations, contact Anatomic SITT AB.

#### Recycling

A worn out part or product should be disassembled and discarded. Separate and sort after material type.

This product is intended to securely position a user. The product should not be used as safety belt during transport, as restraint of a person or in any other manner that could cause injury.

NOTE! The prescriber of the medical device, or someone with equivalent skills and knowledge of the patient and the medical device, must always perform a risk analysis before use. If this analysis shows that there may be risk of serious injury, the patient must never be left unattended when the medical device is used.