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Tri-glide metoden

The Support Series is developed and CE-marked to fit wheelchairs/seating systems on the European Medical Aids Market.


Right direction


Wrong direction

Lower attachments
Zitzi Delfi Pro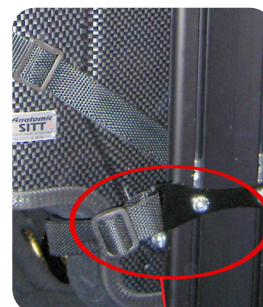

Lower attachments
Zitzi Sharky

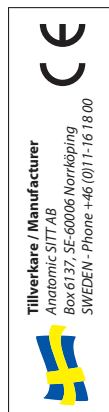- Make sure that there is enough space for the neck.
- The harness **MUST** be used in combination with a hip belt!
- The Sternum support should be mounted by a medical / qualified therapist only.
- The positioning series Support is not intended to replace health care staff supervision.

Mounting on a seating system or a wheelchair:

- 1) First mount the Anatomic SITT positioning SittBelt, (See separate manual), for an optimal correct pelvis position.
- 2) Mount the lower attachments of the strap diagonally downwards with the enclosed Tri-glide buckle.
- 3) Mount the upper attachments in a suitable place with the enclosed buckles:

Alt. 1: When mounting on the stable back - Screw on the enclosed straplock

Alt. 2: When mounting around the frame tubes - Run the strap through the enclosed Tri-glide buckle

Alt. 3: When mounting the "hammock back" - Use the "holder between handles", item 7022 + 3719

The pull direction of the upper attachments:

The direction of the upper attachments should be at the same level as the shoulders to prevent the shoulders from being pressed down. If necessary, use raising stays (Item 3715).

Tri-glide

Adjustment of the Sternum:

- 1) Tighten the upper straps of the Sternum support.
- 2) Make sure that there is enough space for the neck

Supervisory control: Check at every new adjustment:

- the straps directions // - the pressure of the strap // - The space for the neck // - The position of the belt cushions.

Washing instructions: 600, softwash. Do not dry in a tumbler or drying cabinet.

Guarantee: 1 year. Only use spare parts delivered by Anatomic SITT AB.

Check the user continuously to discover any redness of the skin that remains. If such arises, you must immediately stop using the strap.

Wear control: Check the straps, (especially at the fix points), and the belt cushions to discover any signs of wear.

The product fits the seating systems Zitzi Delfi, Zitzi Delfi Pro, Zitzi Sharky & Anatomic SITS. For other combinations, please contact Anatomic SITT AB.

Recycling: A worn out part or product should be disassembled and discarded. Separate and sort after material type.

This product is intended to securely position a user. The product should not be used as safety belt during transport, as restraint of a person or in any other manner that could cause injury.

NOTE! The prescriber of the medical device, or someone with equivalent skills and knowledge of the patient and the medical device, must always perform a risk analysis before use. If this analysis shows that there may be risk of serious injury, the patient must never be left unattended when the medical device is used.